CLASSIC PLUS® Design Thin-Footplate Cervical Kerrison Rongeur

Symmetry Surgical Inc.
3034 Owen Drive
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566
wwwi

EC REP

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany
☎ +49 7461 96490
Fax: + 49 7461 77921

LCN 530-1500-85/J
© 2002–2015 Symmetry Surgical
♻️ Revised 09/1t

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

CLASSIC PLUS® Design Thin-Footplate Cervical Kerrison Rongeur

Indications

This CLASSIC PLUS® Design Thin-Footplate Cervical Kerrison Rongeur is intended for use only on small bone in the cervical area.

WARNINGS

Use of this instrument for purposes or in a manner other than those described here will cause instrument damage or failure that could result in serious patient injury or death.

Inspect instrument before each use.

Do not twist or pry instrument while cutting as this can overstress the tip, leading to possible damage or breakage.

Reprocessing

Refer to Symmetry Surgical Instructions for Use 204233 for updated information regarding reprocessing the instrument.

Copies of this instruction sheet are available by requesting IFU 204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000

Fax: +1 615 964 5566

E-mail:

customerservice@symmetrysurgical.com

Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical i
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

CLASSIC PLUS® Design is a registered trademark of Symmetry Surgical

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