HARDY™ Bivalve Speculum

Symmetry Surgical
3034 Owen Drive,
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566
www.symmetriesurgical.com

[EC REP]

Symmetry Medical Sheffield
Beulah Road
Sheffield, England
S6 2AN
☎ +44-114-285-5881
Fax: +44-114-233-6978

LCN 800-1350-85/I
© 1999–2012 Symmetry Surgical
Revised 07/13

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

HARDY™ Bivalve Speculum

Indications

The HARDY Bivalve Speculum provides transnasal access to the sphenoid sinus during a transphenoidal approach to the pituitary fossa.

Description

Catalog No.	Description
80-1350	HARDY Bivalve Speculum, 7 cm, Small For use on a child or an adult patient with a small nose
80-1351	HARDY Bivalve Speculum, 9 cm, Acromegalic For use on a patient with a large nose or an acromegalic patient
80-1352	HARDY Bivalve Speculum, 8 cm, Standard For use on a patient with a normal size nose

WARNINGS

Usually the bivalve nasal speculum can be opened gently without effort by merely turning the knob. When overpressure is required to open the valve (speculum), it is due to a narrow, bony nasal orifice that must be enlarged by ronguering the lateral ascending branches of the maxilla. Otherwise, overpressure may fracture the maxilla.

Occasionally in acromegalic patients with large turbinates, opening the blades can be facilitated by the use of a spreader.

Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use 204233 for updated information regarding reprocessing the instrument.

Copies of this instruction sheet are available by requesting IFU 204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000

Fax: +1 615 964 5566

E-mail:
ssi-customerservice@symmetrysurgical.com

Storage

Store sterile packaged instruments in a manner that provides protection from dust, moisture, insects, vermin, and extremes of temperature and humidity.

Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed.** Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.

™ HARDY is a trademark of Symmetry Surgical

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Téléphone : 800 251 3000

Fax : +1 615 964 5566

E-mail :
ssi-customerservice@symmetrysurgical.com

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