# KARLIN™ Micro Knife Blade

**Symmetry Surgical**  
3034 Owen Drive,  
Antioch, TN 37013 USA  
☎ 1-800-251-3000  
FAX: 1-615-964-5566

Symmetry Medical Sheffield  
Beulah Road  
Sheffield, England  
S6 2AN  
☎ 011-44-114-285-5881  
Fax: 011-44-114-233-6978

CE  
0086

**LCN 170929-001/M**

© 2002–2013 Symmetry Surgical

♻ Revised 03/13

# ENGLISH

## IMPORTANT INFORMATION

Please Read Before Use

### KARLIN™ Micro Knife Blade

STERILE	R
---------	---

<b>R</b> x Only
-----------------

#### Description

The KARLIN Micro Knife Blades come in two sizes: micro tip (catalog no. 46-3177) and ultra tip (catalog no. 46-3178). The blades are sterile, disposable, for single use only, and fit into the Micro Knife Handle (catalog no. 46-3174).

#### WARNING

The blade is for single use only. Do not resterilize it.

Ensure the blade is stationary in the knife handle before using.

#### CAUTIONS

Do not use pliers or other mechanical assistance to insert or secure the blade in the knife handle.

Be aware that the blade is extremely sharp.

Do not exert any form of sideways pressure on the blade; the blade may deform or break.

#### Assembly

To insert the blade into the Micro Knife Handle (catalog no. 46-3174), turn the collet knob counterclockwise. Insert the blade as far as you require. Tighten the collet knob by turning it clockwise.

## Sterility

2

This product is for SINGLE USE ONLY; DO NOT RESTERILIZE. Use aseptic technique in all phases of handling. Symmetry Surgical Single Use devices have not been designed to undergo or withstand any form of alteration, such as disassembly, cleaning or re-sterilization, after a single patient use. These devices are intended to come into contact with the central nervous system and the ability does not currently exist to destroy possible contaminants such as Creutzfeldt-Jakob Disease. Reuse can also compromise device performance and any usage beyond the design intent of this single-use device can result in unpredictable use hazards or loss of functionality.

Symmetry Surgical will not be responsible for any product that is resterilized, nor will accept for credit or exchange any product that has been opened but not used.

As long as the individual package of the product is not opened or damaged, the product is sterile.

## Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

<sup>TM</sup> KARLIN is a trademark of Karlin Technologies, Inc. (Patents Pending)

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