# Symmetry Surgical™ Crank Frame Spine Retractor Set

---

**Symmetry Surgical**  
3034 Owen Drive,  
Antioch, TN 37013 USA  
☎: 1-800-251-3000  
Fax: 1-615-964-5566

EC REP

Symmetry Medical Sheffield & Cheltenham  
Unit 1, Stella Way Malven View Park  
Bishops Cleeve,  
Cheltenham, GL52 7DQ UK

**LCN 170079-001/H**  
© 2002–2012 Symmetry Surgical  
♻️ Revised 07/12

## ENGLISH

### IMPORTANT INFORMATION

Please Read Before Use

### Symmetry Surgical™ Crank Frame Spine Retractor Set

#### Indications

The Symmetry Surgical Crank Frame Spine Retractor Set (catalog no. 46-3190) is to be used for posterior lumbar and thoracic spine surgery. Its design permits the blades to be prepositioned in the wound prior to attachment and offers ease of assembly, insertion, retraction and stability. Multiple blades, which vary in design, depth, and width, provide versatility to the retractor's use.

#### Description

The set consists of catalog nos. 46-3191 through 46-3197, which are also available separately.

| Catalog no.                                    | Description                                             | Width & Depth |              |
|------------------------------------------------|---------------------------------------------------------|---------------|--------------|
|                                                |                                                         | (approx. in.) | (approx. mm) |
| 46-3191                                        | Crank Frame Retractor Body                              |               |              |
| 46-3192                                        | Crank Frame Retractor Ratchet                           |               |              |
| 46-3193                                        | Crank Frame Retractor Blade, Left, with Sacral Cut-Out  | 1.25 x 1.50   | 32 x 38      |
| 46-3194                                        | Crank Frame Retractor Blade, Right, with Sacral Cut-Out | 1.25 x 1.50   | 32 x 38      |
| 46-3195                                        | Crank Frame Retractor Blade, Left, with Sacral Cut-Out  | 2.25 x 1.75   | 57 x 44      |
| 46-3196                                        | Crank Frame Retractor Blade, Right, with Sacral Cut-Out | 2.25 x 1.75   | 57 x 44      |
| 46-3197                                        | Crank Frame Retractor Blade, Left and Right             | 3.00 x 2.50   | 76 x 64      |
| The following blades are available separately: |                                                         |               |              |
| 46-3199                                        | Crank Frame Retractor Blade, Left and Right, Offset     | 0.75 x 1.25   | 19 x 32      |

|         |                                                     |             |          |
|---------|-----------------------------------------------------|-------------|----------|
| 46-3200 | Crank Frame Retractor Blade, Left and Right, Offset | 0.75 x 1.75 | 19 x 44  |
| 46-3201 | Crank Frame Retractor Blade, Left and Right, Offset | 0.75 x 2.25 | 19 x 57  |
| 46-3202 | Crank Frame Retractor Blade, Left and Right, Offset | 0.75 x 3.00 | 19 x 76  |
| 46-3203 | Crank Frame Retractor Blade, Left and Right, Offset | 0.75 x 3.25 | 19 x 83  |
| 46-3204 | Crank Frame Retractor Blade, Left and Right, Offset | 0.75 x 4.00 | 19 x 102 |
| 46-3205 | Crank Frame Retractor Blade, Left and Right, Offset | 2.00 x 4.00 | 51 x 102 |
| 46-3206 | Crank Frame Retractor Blade, Semi-Hemi              | 1.25 x 1.25 | 32 x 32  |
| 46-3207 | Crank Frame Retractor Blade, Semi-Hemi              | 2.00 x 1.25 | 51 x 32  |

#### WARNING

Using this instrument for purposes or in a manner other than those described can result in instrument damage or failure, which can cause serious injury or death.

#### CAUTIONS

Always clean and sterilize the product before each use, including initial use.

Only use the supplied ratchet wrench to open and close the retractor body.

Always use a water-soluble lubricant on the product after cleaning and before sterilization.

Remove excess lubricant before sterilizing the product.

Inspect the product before each use. Do not use the product if you suspect it may be damaged. Within the United States, send the instrument to Symmetry Surgical Repair Service for service and repair. Outside the U.S., contact your local Symmetry Surgical representative.

Do not expose the product to acids, harsh detergents, or prolonged exposure to saline.

## Insertion

1. Place the desired blades into the incision. The blades may, but are not required to, touch (Figure 2).

2. With the retractor body arms closed, place the retractor body arms between the blade attachments. Be sure that the arm's "step" (Figure 3) faces upward and the blade teeth point toward the desired retraction.

3. Insert the ratchet wrench into the gear box (ratchet) and turn the ratchet wrench clockwise to open the frame (Figure 4). Each incremental "click" opens and locks the retractor in a new secure position.

4. Remove the ratchet wrench for optimal visualization and access (Figure 5).

**NOTE:** Up to three blades can be placed on each retractor arm.

## Removal

1. Reinsert the ratchet wrench and press the lock release mechanism.

2. Keep the lock release mechanism pressed with one hand and turn the wrench counterclockwise with the other hand to close the retractor.

3. Withdraw the retractor body from the blades.

4. Remove the blades from the incision.

**NOTE:** If the ratchet lock does not release, continue to apply pressure to the locking mechanism and turn the ratchet wrench clockwise until the locking mechanism is released. Then turn the ratchet wrench counterclockwise to close the retractor.

**NOTE:** The retractor body also can be withdrawn from the blades by sliding it out from within the blades.

## Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing the instrument. SEE NOTES BELOW.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000

Fax: +1 615 964 5566

E-mail:

ssi-customerservice@symmetrysurgical.com

**NOTE:** Do not use mineral oil, machine oil, or silicone spray as lubricants.

## Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical  
3034 Owen Drive,  
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

## Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

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