Symmetry Surgical Disposable Vein Stripper

Symmetry Surgical
3034 Owen Drive
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566

u u

Symmetry Medical Sheffield
Beulah Road
Sheffield, England
S6 2AN
☎ +44-114-285-5881
Fax: +44-114-233-6978

CE
0086

LCN 182040-001/P
© 2001–2013 Symmetry Surgical
♻ Revised 0 /13

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

Symmetry Surgical Disposable Vein Stripper

STERILE

EO

The lot number and the date are located on the clear plastic side of the package.

Description

The Symmetry Surgical Disposable Vein Stripper, catalog no. 63-4031, consists of the following nylon, x-ray detectable, components:

- 2 Cables 36 in. (91.44 cm) long
- 1 Olive $\frac{3}{8}$ in. (0.95 cm) diameter
- 1 Olive $\frac{1}{2}$ in. (1.27 cm) diameter
- 1 Olive $\frac{5}{8}$ in. (1.59 cm) diameter
- 1 Handle

Indications

The Symmetry Surgical Disposable Vein Stripper is designed for the removal of saphenous varicose veins.

WARNINGS

Do not strip through a tourniquet or other restriction. Cable can break, leaving components in the wound.

Make sure the cable is fully engaged and seated in olive before stripping. Failure to do so could result in the olive becoming detached during stripping.

After initiation of stripping, do not attempt to reverse direction by pushing on cable. Doing so can cause the olive to dislodge from the cable.

Do not use excessive force during stripping. Cable can break, leaving components in the wound.

PRECAUTIONS

Inspect the sterile package carefully.

Do not use if:

- the package or seal appears damaged,
- contents appear damaged, or
- the expiry date has passed.

Instructions for Use

Either end of the cable can be inserted into either end of the vein. Three olive sizes are included to accommodate different vein sizes. The olive and handle are placed on the cable after it has been inserted. The stripper with the vein on it can be sent directly to pathology.

Olive Assembly

1. Insert cable into slot and snap in position. See Figure 1.
2. Draw cable into olive UNTIL IT SEATS. See Figure 2.

Handle Assembly

1. Insert cable into slot and snap in position. See Figure 3.
2. Draw cable into handle UNTIL IT SEATS. See Figure 4.

Sterility

②

This Symmetry Surgical Disposable Vein Stripper is for SINGLE USE ONLY; DO NOT RESTERILIZE. Use aseptic technique in all phases of handling. Symmetry Surgical Single Use devices have not been designed to undergo or withstand any form of alteration, such as disassembly, cleaning or resterilization, after a single patient use. Reuse can potentially compromise device performance and any usage beyond the design intent of this single-use device may result in unpredictable loss of functionality.

Symmetry Surgical will not be responsible for any product that is resterilized, nor will we accept for credit or exchange any product that has been opened but not used.

As long as the individual package is not opened or damaged, the product is sterile.

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

FRANÇAIS

INFORMATIONS IMPORTANTES

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Tire-veine jetable Symmetry Surgical

STERILE	EO
---------	----

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