# Symmetry Surgical™ MAGRINA-BOOKWALTER® Pelvic Retractor Kit

Developed in cooperation with Javier F. Magrina, M.D.

---

---

**Symmetry Surgical**  
3034 Owen Drive,  
Antioch, TN 37013 USA  
☎ 1-800-251-3000  
Fax: 1-615-964-5566  
www.A A

EC REP

Symmetry Medical Sheffield  
Beulah Road  
Sheffield, England  
g S  
☎ 8cc: NNc: Sl e: el I N  
o8cc: NNc: Saa: gmi l

**LCN 206060-001/D**  
© 2008–2012 Symmetry Surgical  
♻ Revised 07/1a

## ENGLISH

### IMPORTANT INFORMATION

Please Read Before Use

## Symmetry Surgical™ MAGRINA-BOOKWALTER® Pelvic Retractor Kit

Developed in cooperation with  
Javier F. Magrina, M.D.

### Description

The Symmetry Surgical MAGRINA-BOOKWALTER Pelvic Retractor Kit is a self-retaining retraction system. The Kit is available in two configurations. Catalog no. 50-4692 (Figure 1) includes a segmented pelvic ring, while catalog no. 50-4693 contains a non-segmented pelvic ring. All components of the kit are fabricated of stainless steel. The pelvic retractor kit contains the following instruments, each available separately:

- A. BOOKWALTER Table Post 1 in. (2.5 cm) x 8 in. (20.3 cm) (1 each) catalog no. 50-4683
- B. BOOKWALTER Horizontal Flex Bar (1 each) catalog no. 50-4582
- C. BOOKWALTER Post Coupling 1 in. x 1 in. (2.5 cm x 2.5 cm) (1 each) catalog no. 50-4554
- D. Symmetry Surgical MAGRINA-BOOKWALTER Segmented Pelvic Ring (1 each) catalog no. 50-4689 or non-segmented pelvic ring (1 each) catalog no. 50-4690
- E. BOOKWALTER Super Malleable Blade 1 in. (2.5 cm) x 4 in. (10.2 cm) (2 each) catalog no. 50-4691
- F. BOOKWALTER Malleable Blade 1 in. (2.5 cm) x 4 in. (10.2 cm) (2 each) catalog no. 50-4694
- G. Vaginal Retractor Lateral Blade (1 each) catalog no. 50-4591
- H. Vaginal Retractor Deaver Blade Small (1 each) catalog no. 50-4592
- I. Vaginal Retractor Deaver Blade Large (1 each) catalog no. 50-4593
- J. Vaginal Retractor Posterior Blade (1 each) catalog no. 50-4594
- K. BOOKWALTER Tilt Ratchet Mechanisms (6 each) catalog no. 50-4580

### Indications

The pelvic retractor kit is designed for retraction in vaginal surgery.

### WARNING

The retractor system can exert a great amount of force on the wound edges and on tissues. Release the retractor for short periods of time or shift the blades to relieve the pressure on any one point.

If the table is to be tilted toward the side holding the post, it is important to place adequate padding between the table post and the patient. Pressure necrosis or a nerve palsy could result if the patient is pressed against the unpadded bar for a long period.

### Precautions

If using monopolar electrosurgical equipment while using the Symmetry Surgical MAGRINA-BOOKWALTER Pelvic Retractor Kit, consult the basic rules of usage concerning electrosurgery (see EN 60 601-2-2), as well as the manufacturer's instructions for use regarding specific warnings and conditions.

Be sure to electrically isolate the patient from (high frequency leakage currents to) the operating table, in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Ensure that the patient does not come into contact with metal parts that are earthed or that have an appreciable capacitance to earth (for example: operating table, supports), in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Avoid small-area contacts with objects having a low impedance to earth at high frequencies, as unwanted burns can result.

### Inspection

Inspect each instrument before and after use to detect imperfections. Do not use damaged instruments; send them for repair or replacement.

### Instructions for Use

#### Attachment of the Table Post

Pull out the recessed lever located at the top of the table post (Figures 2 and 3), and fully open the rail clamp by turning the lever in a counterclockwise direction. Attach the clamp over the drapes on the draped side rail; the table post will be parallel to the floor (Figure 4). Retighten the clamp to secure the post by turning the lever in a clockwise direction.

**NOTE:** Be sure the operating table foot rail does not have excess material draped over it. The clamp of the table post is designed to hold no more than two double thicknesses of cotton drapes.

### **Attachment of the Post Coupling, Horizontal Flex Bar and Pelvic Retractor Ring**

**NOTE:** For convenience and correct orientation of the ring, it is recommended that the pelvic ring be positioned with the 20 degree break at the level of the posterior vaginal wall while the patient is in the high lithotomy position.

Slide the horizontal flex bar into either of the holes in the post coupling. Slide the post coupling and horizontal bar assembly over the top of the table post, to its approximate location, and lightly tighten the coupling with the post coupling lever (Figure 5). The position of the post coupling lever on the table post must be convenient to the surgeon. Make any final positioning adjustments before tightening the post coupling.

Place the pelvic retractor ring into the vise clamp of the horizontal flex bar. Tighten the knob of the clamp to secure the ring at the 5 or 7 o'clock position on the horizontal flex bar (Figure 6). This will depend on whether the table post is attached to the right or left side of the surgeon.

**CAUTION:** The serrated teeth of the ring must slide past the lips of the clamp on the horizontal flex bar to ensure a proper fit (Figure 7).

Loosen the post coupling lever and reposition the horizontal flex bar in order to center the ring over the operative site (Figure 8). Make sure the serrated teeth of the post coupling are fully disengaged before attempting any movement. Hold the ring in position and tighten the post coupling lever to lock in place. Make any final positioning adjustments with the ring applied over the vulva.

**CAUTION:** Never tighten the post coupling unless it is attached to the table post and the horizontal flex bar. Tightening the post coupling while not attached to these bars can cause damage to the instrument.

### **Adjustment of the Angle of the Pelvic Retractor Ring (catalog no. 50-4690)**

To adjust the angle of the pelvic retractor ring, release the universal joint lever on the horizontal flex bar (Figure 9) by turning the lever in a counterclockwise direction. Relaxing the universal joint assembly results in a variety of positioning options. Retighten the lever by turning clockwise.

### **Adjustment of the Angle of the Segmented Pelvic Retractor Ring (catalog no. 50-4689)**

**NOTE:** When removing or adjusting the segmented pelvic retractor ring, turn the locking screw counterclockwise until the serrated teeth of the fittings disengage. Adjust the segment as required and retighten the locking screw.

**CAUTION:** Before fully tightening the locking screw, always be certain that the serrated teeth of the ring segment fittings mesh. Failure to do so can damage fittings. To remove the screw from a ring segment for cleaning, pull the screw through the fitting until it engages the threads. Unscrew and remove by turning clockwise.

### **Attachment of the Retractor Blade**

Attach the retractor blade to a tilt ratchet mechanism by inserting the retractor blade handle through the rectangular opening in the ratchet mechanism. Make sure the toothed side of the blade handle faces the ratchet release so that advancement of the blade generates a clicking sound.

**NOTE:** The ratchet mechanisms are designed to fit on any retractor blade.

Grasp the vagina with the retractor blade and pull the blade until the tilt ratchet mechanism connects onto the pelvic retractor ring. Figure 10 depicts connection of the tilt ratchet to the ring.

To adjust the blade, press in the ratchet release and push or pull blade to desired setting. To use the lifting capability of the tilt ratchet mechanism, simply depress the tilt ratchet thumb lever and adjust the elevation.

**WARNING:** The retractor system can exert a great amount of force on the wound edges and on tissues. Release the retractor for short periods of time or shift the blades to relieve the pressure on any one point.

### **Reprocessing Instructions**

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing of the instrument. See CAUTIONS below.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000

Fax: +1 615 964 5566

E-mail:

ssi-customerservice@symmetrysurgical.com

**CAUTION:** If flash autoclaved, take special care to cool off the vertical table post, horizontal flex bar, and post coupling with sterile, distilled water. These items will retain heat and require cooling before assembly.

**CAUTION:** To ensure proper function do not lubricate the Horizontal Flex Bar catalog no. 50-4582.

### Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical  
3034 Owen Drive,  
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

### Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

® BOOKWALTER is a registered trademark of Symmetry Surgical

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