OPTI-LENGTH® Rongeurs

Symmetry Surgical Inc.
3034 Owen Drive
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566
www.symmetrysurgical.com

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany
☎ +49 7461 96490
Fax: +49 7461 77921

LCN 197439-001/I ©
2002–2015 Symmetry Surgical
♻️ Revised 06/15

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

OPTI-LENGTH® Rongeurs

Indications

Cervical Rongeur:

Specifically designed bone nipping device for cervical spine reconstruction.

IVD Rongeur:

Specifically designed for removing intervertebral disc material during laminectomy procedures.

Laminectomy Rongeur:

Specifically designed bone nipping device for laminectomy procedures.

Bone Rongeur:

Specifically designed for rongeur small bone and for minor orthopaedics.

Offset Rongeur:

Primarily designed for microsurgery to rongeur soft tissue, such as ligaments, and to rongeur small bone edges and spicules in spinal or cranial surgery.

IMPORTANT: SEE PACKAGE LABEL FOR A DESCRIPTION OF THE CONTENTS.

WARNINGS

Instruments are specifically designed for use as indicated above. Use of this instrument for unintended purposes or in a manner other than those described here will cause instrument damage or failure. Instrument failure can result in dissemination of metal pieces in the operative area, possibly causing serious and permanent injury including, but not limited to, damage to surrounding tissue and/or structures.

The rongeur is designed to make high precision cuts. Inspect the cutting edges of the rongeur before and after each use for sharpness. Ensure that there are no dull edges, no deformations and no rollover on the cutting edges. If any of these appear, send it for repair or resharpener. Failure to do so can result in imprecise bite size and difficulties in nipping bone. Continued use can permanently damage the instrument.

Do not remove crossbar screw from the rongeur; increased risk of screw backout and subsequent device failure could result.

Do not use Cervical rongeurs in the lumbar or thoracic areas. Incorrect use can result in instrument breakage and the potential dissemination of instrument fragments into the wound.

Do not use IVD rongeurs to rongeur bone. Do not use IVD rongeurs in a twisting or prying motion. Incorrect use can result in instrument breakage and the potential dissemination of instrument fragments into the wound.

Use Bone rongeurs with care on large spinal and cranial bone. Make several small cuts to achieve the correct sized bite. Twisting and/or prying motions can result in breakage and the potential dissemination of instrument fragments into the wound.

Do not use Offset rongeurs for heavy bone work, as they are delicate and can break when used incorrectly. Twisting and/or prying motions can result in breakage and the potential dissemination of instrument fragments into the wound.

Inspection

Inspect all instruments prior to sterilization or storage to ensure instruments are suitable for use. Disassemble instruments, as appropriate and inspect for damage. Set aside cracked or bent instruments; send these instruments for service or repair.

Note: Do not remove crossbar screws. Doing so can lead to screw back-out and subsequent equipment failure.

Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing of the instrument.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000

Fax: +1 615 964 5566

E-mail:

ssi-customerservice@symmetrysurgical.com

Service and Repair

For service and repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

OPTI-LENGTH® is a registered trademark of Symmetry Surgical

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