OPTI-LENGTH[®] RAPIDCLEAN[®] Detachable Rongeur

RAPIDCLEAN[®] Detachable Rongeur

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566
www.e

EC REP

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany

☎ Fry urtl ytryi
Fry urtl uuynt

LCN 198230-001/M

© 2002–2015 Symmetry Surgical
♻ Revised 09/15

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

OPTI-LENGTH® RAPIDCLEAN® Detachable Rongeur

RAPIDCLEAN® Detachable Rongeur

Description

The OPTI-LENGTH RAPIDCLEAN and RAPIDCLEAN Detachable rongeurs are designed for rongeur bone during spinal procedures. The crossbar of the rongeur detaches completely from the body of the instrument for cleaning. Cervical and lumbar rongeurs are available in varying lengths and bite sizes.

The rongeur is provided nonsterile and must be cleaned and sterilized before use, including initial use.

Indications

Cervical rongeur:

Indicated for bone-nipping in cervical spine reconstruction procedures.

Lumbar rongeur:

Indicated for rongeur bone in lumbar spine procedures.

WARNINGS

Instrument failure can result in dissemination of metal pieces in the operative area, possibly causing serious and permanent injury including, but not limited to, damage to surrounding tissue and/or structures.

The rongeur is designed to make high-precision cuts. Inspect the cutting edges of the rongeur before and after each use for sharpness. Ensure that there are no dull edges,

no deformations and no rollover on the cutting edges. If any of these appear, send it for repair or resharpening. Failure to do so can result in imprecise bite size and difficulties in nipping bone. Continued use can permanently damage the instrument.

Do not use cervical rongeurs in the lumbar or thoracic areas. Incorrect use can result in instrument breakage and the potential dissemination of instrument fragments into the wound.

Precautions

Do not interchange the crossbar of one detachable rongeur with the crossbar of any other detachable rongeur. Always make sure the control number on the rongeur crossbar matches the control number on the bottom-half of the rongeur when reassembling (see Figure 4). If you attempt to assemble a rongeur with non-matching control numbers, you can damage both pieces of the instrument.

Do not drop or throw the rongeur. This can break or warp the rongeur. If the rongeur is dropped or thrown, disassemble and closely inspect it to make sure there is no breakage or damage.

When reassembled, the crossbar must move freely along the bottom half. If it binds or sticks, disassemble the instrument and check for cleanliness and lubrication.

Do not use an instrument or a screwdriver to lift the safety lock or to turn the thumbscrew. Damage to the instrument can occur.

Do not use lumbar rongeurs in the cervical spine.

Disassembly Procedures

Refer to Figures 1 through 3 for disassembly instructions.

Step 1 – Gently turn the thumbscrew counterclockwise until it stops.

CAUTION: Do not attempt to remove the thumbscrew; this will damage the rongeur.

Step 2 – Pull up on the safety lock.

Step 3 – Slide the crossbar to the rear until it stops. Lift the crossbar from the bottom half.

Reassembly Procedures

CAUTION: Clean and lubricate the detachable rongeur before reassembly.

CAUTION: Make sure the control numbers on the crossbar and the bottom half match before reassembling the rongeur.

Refer to Figures 4 through 9 for the detailed procedures for reassembling the detachable rongeur.

Note: Safety lock must be in the up (unlocked) position.

Step 1 – Fully seat the crossbar into the bottom half.

Step 2 – Push the crossbar forward until it stops. The alignment line on the crossbar must be slightly to the rear of the alignment line on the bottom half.

Step 3 – Press the safety lock downward until it snaps into place. Push the crossbar forward again until the alignment lines are lined up.

Step 4 – Gently turn the thumbscrew clockwise until it is finger-tight.

CAUTION: Do not over tighten the thumbscrew. Damage to the instrument can occur.

Step 5 – Test the reassembly to ensure smooth movement by squeezing the handle and quickly releasing it while making sure that the cutting edge of the crossbar comes flush with the footplate. If there is a gap, repeat the reassembly procedure. If the crossbar sticks or binds when the handle is squeezed, disassemble the instrument and check to see that it is clean and lubricated.

Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing of this instrument.

See CAUTIONS below.

Copies of this instruction sheet are available by requesting IFU # 204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000

Fax: +1 615 964 5566

Email:

ssi-customerservice@symmetrysurgical.com

CAUTIONS:

Disassemble the rongeur before cleaning. Lubricate the disassembled rongeur after every cleaning. Failure to follow these instructions can result in damage to the rongeur, such as sticking and rough cutting action.

To prevent damage to the rongeur, do not use metal or stiff bristle brushes when cleaning.

Service and Repair

For service or repair outside of the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

® OPTI-LENGTH and RAPIDCLEAN are registered trademarks of Symmetry Surgical

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